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These condoms are made of natural rubber latex. Caution: Natural rubber latex may cause allergic reactions.

A condom offers reliable protection against pregnancy and sexually transmitted diseases. "Accidents" are usually the result of incorrect use. You should therefore read the instructions for use beforehand.

INSTRUCTIONS

- Only remove the condom from the foil envelope immediately before use.
- Tear the foil at the notch. Do not use sharp objects!
- Watch out for sharp fingernails when putting on the condom.
- Put on the condom when the penis is completely erected. Pulling back the foreskin makes it easier to put the condom on.
- While putting on the condom, pinch the sperm reservoir closed with your fingers. This prevents air getting trapped in the reservoir. If air should get into the sperm reservoir, we advise you to use a new condom.
- Place the condom on the end of the penis and roll down the condom as far as possible.
- After sex, hold down the bottom end of the condom so that it cannot slide off the penis while pulling out.

IMPORTANT

- Never use a condom a second time!
- If extra lubrication is needed, use only water-based lubricants. Oil-based lubricants (vaseline, vegetable oil, etc.) can weaken the condom considerably.
- Dispose of condoms in the bin, never in the toilet, as this can lead to sewage-pipe blockages.
- Do not use the condom after the "expiry date".
- Consult a doctor or pharmacist about the compatibility with condoms of prescription of over-the-counter topical medicines that are applied to the penis or the vagina.

CONDOMS AND THE HIV VIRUS

- With ROMED condoms the risk of pregnancy and/or sexually transmitted diseases is reduced considerably. However, no condom can offer 100% security.

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