(B) Important Information for Easyglide Natural Rubber Latex Condoms, please read carefully. Always handle the condom carefully including removal from the package so as to avoid damage to the condom by fingernails, jewellery, etc. If condom is damaged discard that condom and use a new undamaged one. Condoms made from natural rubber latex may cause allergic reactions including anaphylactic shock. A condom should never be used for a second time, condoms are for single use and if reused there might be an increased risk of STI or infection, the risk of breakage is also higher. The use of condoms cannot guarantee 100% protection against pregnancy, HIV, or Sexually Transmitted Infections. If in doubt always consult a doctor. If additional lubricant is desired, use the type of lubricant recommended for use with condoms e.g. for Natural Rubber Latex Condoms a water-based lubricant. Avoid the use of oil based lubricants, such as petroleum jelly, baby oil, body lotions, massage oils, butter, margarine, etc. as these are deleterious to the integrity of the latex condom. Use of condoms during anal sex increases the risk of breakage. It is recommended to use additional lubrication if vaginal lubrication is insufficient or for anal sex to minimize the risk of condom breaking or coming off. Consult a doctor or pharmacist about the compatibility of topical medicines that might come in contact with the condom. Carrying condoms in your wallet, back pocket, handbag, or similar is not advisable. If you have unprotected sex or your method of contraception has failed, contact your Sexual Health Clinic or GP for emergency contraception advice preferably within 72 hours. Keep away from children, children can choke or suffocate on uninflated or broken condom fragments. Use of condom during oral sex increases the risk of asphyxiation, choking or condom breaking. A latex condom is a medical device. The device is intended to be used by adult lay persons. Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established. Store in a cool, dry place away from direct sunlight.

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The image shows a blue rectangular box with white text. The text reads "INSTRUCTIONS FOR USE" on the top line and "CONDOMS" on the bottom line, both oriented vertically. To the left of the box is the "easyglide" logo, which consists of a stylized blue 'e' shape followed by the brand name in a blue, lowercase, sans-serif font.

**INSTRUCTIONS FOR USE
CONDOMS**

EASy IDE in a consistent command

EASYGLIDE is a registered trademark.
Produced in license by EQOM Purchasing B.V.
Designed in The Netherlands. Made in Malaysia.
Phoenixweg 6, 9641 KS, Veendam NL (EU).
Customer service: +31 (0)598 - 69 04 53
www.easyglide.eu | info@easyglide.eu

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3. Hold the condom tight between your thumb and finger making sure that the condom is the right way for unrolling (the roll must be outside).

4. Squeeze the test and wait until the condom is in place. After use, remove the condom from the vaginal and anus.

5. After use make sure you dispose of the condom hygienically. Wrap it in a tissue and place it in the trash. *Do not flush it down the toilet.

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