einhorn EXTRA BLATT

WAS GEHT'S HIER??

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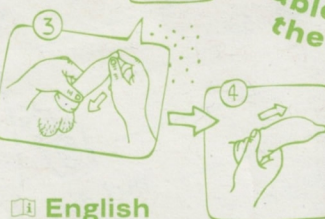
1 English

Let the magic begin!

The condom should be put on before any sexual contact. That's the only way to protect yourself from sexually transmitted diseases and unwanted pregnancy. And, you know, properly enjoy the next few minutes or hours (whoa yeahh!).

1 Please be very careful when removing the condom from the packaging. Fingernails, jewelry, or hedgehogs don't belong anywhere near this sensitive sheath. Condoms are robust and resilient, but not indestructible.

2 Pull the foreskin back on the erect penis and place the condom right side up on the tip.

3 Pinch the reservoir with your thumb and index finger to keep air out and roll it all the way down without tugging. Did you put it on inside out? Grab another one - there's already bodily fluid on the outside.

4 Ooh... done? Pull the still-erect penis out while holding the condom in place to avoid any leakage of sperm.

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The intestinal bacteria can lead to inflammation inside the vagina, so you better have several condoms with you.

Oh no!

Did the condom slip off, rip or burst? There's only one thing to do: visit the doctor as soon as possible within 72 hours. Find more on: www.einhorn.my/OhNo

BTW

Latex condoms are made from natural rubber and can be damaged by direct contact with medicinal products like ointments. Ask your pharmacist or doctor before using them. Some people are allergic to latex, which can cause skin irritation. Even if this is rare, it can cause anaphylaxis. Use latex-free condoms if you are allergic.

Einhorn condoms offer effective protection against pregnancy and sexually transmitted infections or viruses. But just like all other contraception methods, 100% safety can't be guaranteed.

Just like the mythical unicorn they are named after, einhorn condoms are notoriously sensitive creatures. Protect them from extreme heat and sunlight, as well as moisture.

Safe and sound?

Does the individual container look damaged or

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www.einhorn.my/geileskondom

Is the expiration date a bit ancient? Even if the rubber material is sticky, brittle, torn or obviously damaged, do not use it and take another condom from a new pack. Always check the expiration date.

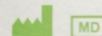
Sexually transmitted infections

Those who haven't had one (congrats) or believe they haven't, often don't know what kind of infections there are. It's definitely worth knowing a bit more about it. www.einhorn.my/liebernich

einhorny latex

The latex used for our condoms is 100% sourced from rubber trees that are planted in biodiverse agroforestry systems. Joining forces with our partners in the value chain, we created the „Regenerative Rubber Initiative“.

We aim for collaboration not competition, and share our latex with other brands - because sharing is caring. www.einhorn.my/geileskondom

Richter Rubber Technology
Sdn. Bhd. 09300, Kuala Ketil
Kedah, Malaysia
www.myrtr.de

Designed & distributed in Berlin
by einhorn products GmbH
horny@einhorn.my

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CE
1639
ISO 4074:2015

231201 REV0

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Service GmbH