ES

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EN

Open the condom foil carefully. Take care not to damage the condom with fingernails, rings or body jewelry.

Do not unroll the condom before putting it on. Unrolling the condom before use may damage it and will make it more difficult to put on. Put the condom on before any contact occurs between the penis and the partner's body.

Gently hold the reservoir tip of the condom between thumb and forefinger (this makes sure there is no air trapped in it) and unroll the condom onto the erect penis, rolling smoothly all the way down. After you have put on the condom, add plenty of lubricant if desired; take care only to use condom-friendly lubricants because others might damage the condom.

Withdraw the penis from your partner soon after ejaculation, before the erection goes down. Hold the rim of the condom round the penis so that the condom does not slip off during withdrawal; this should prevent any spillage.

After withdrawal, take care not to touch the vaginal area with either the penis or the condom.

If the condom breaks or slips off during intercourse, do not ejaculate inside your partner and withdraw immediately.

Check with a doctor or a pharmacist about the compatibility of condoms and topical medicines applied to the penis, vagina or rectum.

Dispose of used condoms by wrapping and putting in a garbage bin. Do not flush down toilet.

!

Storage: Store condoms in a cool dry place away from direct sunlight. Do not use the condom after the use-by date indicated on the foil or package.

Side-Effects and Incompatibility: Condoms are a mechanical barrier and do not interfere with your organism. However, condoms made from natural rubber latex should not be used by people having an allergy or cross-allergy against latex or whose skin reacts or aches when getting in touch with latex. If in doubt, consult your physician as soon as possible and use non-latex condoms.

Prevention of Pregnancy and STI Transmission: Although no contraceptive can guarantee 100% effectiveness, condoms when properly used can help prevent pregnancy and reduce the risk of AIDS and the spread of many sexually transmitted infections (STIs), but cannot totally eliminate the risk of acquiring these diseases. STIs as the name suggests are usually contracted as a result of sexual contact. Any person evidencing symptoms should consult their doctor, STI/GUM clinic or hospital as soon as possible.

Serious incidents should be reported to the manufacturer and to the competent authority of the state where the user resides.

This device is marked with a Unique Device Identifier (UDI). You can find the UDI in barcode and human-readable format on the packaging.

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