## English

### Additional information about condoms:

1. Protection: For prevention of pregnancy and sexually transmitted infections.
2. Handling: Handle carefully and avoid damage when opening (e.g., from fingernails or jewelry).
3. If damaged: Use a new condom and dispose of the damaged one.
4. Material check: Do not use if the latex is sticky, brittle, or the packaging is damaged.
5. Replacement: If damaged, use a new container and dispose of the old one.
6. Drug compatibility: Ask a doctor or pharmacist if your medications are compatible with condoms.
7. Lubricants: Use only water-based lubricants. Do not use oil-based products like Vaseline, baby oil, body lotion, butter, etc.
8. Usage: Put the condom on the erect penis before any sexual contact.
9. Slippage: If it slips or feels tight, stop and check for damage.
10. After ejaculation: Withdraw immediately and hold the condom at the base.
11. If torn or punctured: Seek medical help immediately, at the latest within 72 hours.
12. Disposal: Do not flush down the toilet. Dispose of in household waste.
13. Single use: Do not reuse. This increases the risk of failure and infections.
14. Intended use: Regular condoms are intended for vaginal use only – not suitable for anal sex.
15. Allergy warning: Natural rubber latex may cause allergic reactions, including anaphylactic shock.
16. Effectiveness: No contraceptive offers 100% protection against pregnancy, AIDS, or infections.
17. Storage: Store in a cool, dry place away from direct sunlight. Keep out of reach of children (choking hazard).
18. Oral sex: Use during oral sex increases choking risk.
19. Medical device: A condom is a medical device for laypersons – adult men or women.
20. Reporting obligation: Serious incidents must be reported to the manufacturer and the relevant authority. These condoms comply with international standards ISO 4074 / EN ISO 4074.

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