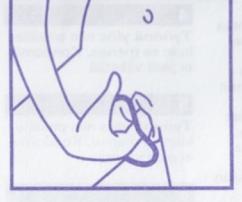
1

2

3

4

5

6

7

IMPORTANT INFORMATION

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6

IMPORTANT INFORMATION

The FC2 Female Condom® is free of natural rubber latex.

A female condom, also known as an internal condom, is designed to be retained in the vagina during sexual intercourse to prevent pregnancy and sexually transmitted infections (STIs).

Store at room temperature in a dry place away from direct sunlight.

No barrier method, including male condoms or female condoms, provide 100% protection against pregnancy and STIs.

Important things to note for the best protection with the FC2:

- Before you try the FC2, be sure to read the directions and learn how to use it correctly.
- Practice putting the FC2 in the vagina without having sex to get familiar with how it works.

Using the FC2:

- The FC2 is inserted before sex. Most people insert the FC2 between 2 to 20 minutes before sex.
- FC2 can be used with most commercially available sexual lubricants.

Risks of using FC2:

- Use the FC2 correctly every time you have sex. If the FC2 is not used correctly every time, your risk of getting a sexually transmitted infection or becoming pregnant will be higher.

7

IMPORTANT INFORMATION

The FC2 Female Condom® is free of natural rubber latex.

A female condom, also known as an internal condom, is designed to be retained in the vagina during sexual intercourse to prevent pregnancy and sexually transmitted infections (STIs).

Store at room temperature in a dry place away from direct sunlight.

No barrier method, including male condoms or female condoms, provide 100% protection against pregnancy and STIs.

Important things to note for the best protection with the FC2:

- Before you try the FC2, be sure to read the directions and learn how to use it correctly.
- Practice putting the FC2 in the vagina without having sex to get familiar with how it works.

Using the FC2:

- The FC2 is inserted before sex. Most people insert the FC2 between 2 to 20 minutes before sex.
- FC2 can be used with most commercially available sexual lubricants.

Risks of using FC2:

- Use the FC2 correctly every time you have sex. If the FC2 is not used correctly every time, your risk of getting a sexually transmitted infection or becoming pregnant will be higher.

Incorrect use may result in the FC2 not providing protection. To avoid risk of incorrect use, read the instructions carefully.

- The following events are rare but can occur when using the FC2:
 - Discomfort during insertion.
 - Pain after insertion, before sex.
 - Discomfort during sex.
 - Burning sensation, rash or itching.

Any serious incident that has occurred in relation to the device must be reported to the manufacturer and the competent authority of the Member State in which the user lives.

Hints

- Add extra lubricant if:
 - Outer ring is pushed inside the vagina.
 - Penis sticks.
 - You hear noise during sex.
 - Be careful of sharp nails and jewellery to avoid tearing the condom.
 - Seek medical assistance as soon as possible, at least within 24 hours, should the female condom leak or burst during use.

Consult a doctor or pharmacist about the compatibility of topical medicines that may come into contact with FC2.

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