6. Important information for PASANTE INFINITY X DELAY Natural Rubber Latex Condoms, please read carefully. 1. This product contains 8.5% benzocaine lubricant. 2. Make sure there is adequate lubrication. If you add lubricant, use a water-based lubricant. DO NOT USE OIL BASED LUBRICANTS such as those with petroleum jelly, mineral oil or cold cream, as these may damage the condom. 3. Some latex products may cause allergic reactions or irritation on contact. If you are unsure on the compatibility of these products with condoms, please consult your doctor or pharmacist prior to use. 4. If the rubber material is sticky or brittle or the individual container is obviously damaged, do not use the condom. 5. Condoms are for single use only. The reused condom could increase the risk of breakage and infection. 6. Seek medical assistance as soon as possible, at least within 72 hours should a condom leak or burst during use. 7. Latex condoms are contraindicated for individuals with a known sensitivity to latex. 8. AIDS and other sexually transmitted infections. 9. If after intercourse, you are concerned that you or your partner may have become pregnant or contracted an STI, consult your doctor immediately. 5. This product contains natural rubber latex which may cause allergic reactions. Includes phytylphoric shock. Some individuals may experience sensitivity to latex products. If you are allergic to latex products, consult your doctor before using this product. If you have a history of allergic to local anaesthetics (i.e. lidocaine, bupivacaine, etc.) or to local anaesthetic "caine" anaesthetics 11. Premature ejaculation may be due to a condition requiring medical supervision. If this product is used as directed and does not provide relief, discontinue the use and consult a doctor. Avoid contact with the eyes. If you or your partner develops a rash or irritation, such as burning or itching, discontinue use immediately. If symptoms persist, consult a doctor. 12. When benzocaine is in contact with the mucous membranes, it may block the transmission of nerve impulses and conduct electrical signals. This blocks the transmission of nerve impulses that carry pain messages. In this way, benzocaine numbs the nerve endings near the surface of the mucous membrane that is in contact with the benzocaine lubricant 13. When used correctly, benzocaine lubricant in the tip of condom will not come in contact with other parts of the body. The benzocaine lubricant can cause numbness if it comes into contact with the mucous membrane after intercourse 15. Keep out of reach of children. 16. Store in a cool dry place away from direct sunlight.

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