Always handle the condom carefully including removal from the package so as to avoid damage to the condom by finger nails, jewellery, etc. Condoms made from natural rubber latex may cause allergic reactions in some individuals. If you are allergic to latex, avoid using latex condoms. If you are not allergic to latex, condoms should never be used for a second time. Condoms are for single use and if reused there might be an increased risk of failure or infection, the risk of breakage is also higher. The use of a condom can guarantee 100% protection against pregnancy, HIV, or Sexually Transmitted Infections (STIs). If in doubt always consult a doctor. If additional lubricant is desired, use the type of lubricant recommended for use with the condom. Do not use oil-based lubricants such as Vaseline[®] or Light Lube and the need to avoid the use of oil based lubricants, such as petroleum jelly, baby oil, body lotions, massage oils, butter, margarine, etc., as these are deleterious to the integrity of the latex condom. Use of condom during and sex increases the risk of breakage. It is recommended to use additional lubricant to reduce the risk of breakage. If you are unsure of the sex to minimize the risk of condom breaking or coming off. Consult a doctor or pharmacist about the compatibility of topical medicines that might come in contact with the condom. Carrying condoms in your wallet, back pocket, handbag, or similar is not advisable. If you have unprotected sex or your method of contraception has failed, contact your doctor or pharmacist for advice. If you are unsure of the advice preferably no later than 72 hours. Keep away from children, children can choke or suffocate on uninflated or broken condom fragments. Use of condom during oral sex increases the risk of asphyxiation, choking or condom breaking. A latex condom is a medical device. The device is intended to be used by adult lay persons. Any serious incident or adverse reaction should be reported to the competent authority to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

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Master reference information: K4FU-2(1)(v.1)

Glow

In order to get Pasante Glow condoms to glow they need to be exposed to light for about 30 seconds. You may expose the condom to light while it is still in the wrapper, when you are putting it on, or after it is already on. For best results, keep the lights on bright while you put on the condom, or turn up the lights for about 30 seconds after you put it on. Once activated, the condom will glow brightly for about 15 minutes, followed by an afterglow that can last for hours. Re-exposing the condom to light while it's still on will recharge it and make it glow brightly again. But remember, you should **never re-use a condom.**

Expose to light for 30 seconds | 30 Sekunden lang Licht aussetzen | Exposer à la lumière pendant 30 secondes | Exporre alla luce per 30 secondi | Exponer a la luz durante 30 segundos | Exponera för ljus | 30 sekunder | Wystaw na działanie światła przez 30 sekund | Vysvetlite na svetlinu za 30 секунд | Vystavte svetlu po dobu 30 sekund | Vystavte svetlu po dobu 30 sekund | Izložite svetlu 30 sekundi | تعريض الحزمة للضوء لمدة 30 ثانية

Pasante Healthcare Ltd.
Commerce Way
Lancing, West Sussex
BN15 8TA
UK
+44 (0) 1903 753844
www.pasante.com