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nulatex®

CE 1639  
ISO 4074

Manufactured by:  
**Nulatex Sdn. Bhd.**  
Lot 1870, 4th Mile, Jalan Mengkibol,  
86000 Kluang, Johor, Malaysia.

EC REP

**Advena Ltd.**  
Tower Business Centre, 2nd Fl.,  
Tower Street, Swatar, BKR4013 Malta

NSB-L003 Rev 02  
NAWL013/02 Effective Date: 20 July 2020

GB

## INSTRUCTION FOR USE-NATURAL RUBBER LATEX MALE CONDOM

**Caution:** THIS PRODUCT CONTAINS NATURAL RUBBER LATEX WHICH MAY CAUSE ALLERGIC REACTION INCLUDING ANAPHYLACTIC SHOCK.

### List of dos and don'ts to ensure you use and care for your condoms properly:

- Don't use a condom past its expiry date as rubber latex deteriorates with age.
- Do take off the condom immediately after sex. Wrap the used condom with a tissue paper and dispose it properly. Do not flush used condoms down toilet bowl.
- Consult a doctor or pharmacist about the compatibility of topical medicines that may come in contact with the condom.
- During sex, if you feel the condoms slipping off or tightening excessively onto the penis, please stop and check as this might lead to breakage
- Seek medical assistance as soon as possible, at least within 72 hours should a condom leak or burst during use.

### Precaution

- Use a new condom every time you have sexual intercourse or other acts between partners that involve contact with the penis.
- DO NOT REUSE Condoms
- Store condoms in a cool, dry place away

from direct sunlight and heat. Heat can deteriorate the physical properties of condom.

- If the rubber material is sticky or brittle or obviously damaged, do not use it.
- If additional lubricant is required, use water-based lubricants. DO NOT USE LUBRICANTS, such as those made with petroleum jelly, mineral oil, vegetable oil, or cold cream.
- If the individual container is damaged do not use it. Use a new one from an undamaged package.

### Instruction for use

**Intended use:** To prevent pregnancy (contraception) and reduce the risk of the transmission of sexually transmitted infections (STIs).

1. Take your condom out of the package carefully, making sure that you don't damage it with fingernails or jewellery.
2. Place the condom on the head of the penis and unroll or pull it all the way to the base.
3. Leave an empty space at the end of the condom to collect semen. Remove any air remaining in the tip of the condom by gently pressing the air out toward the base of the penis.
4. After ejaculation and while the penis is still erected, hold onto the rim of the condom so that the condom does not slip off the penis when withdrawing.

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